

B. Graczyk, Inc. Universal™ / Surgical Eye Care Products

Mixing Universal Concentrated Surgical Instrument Cleaner and Lubricant for Use:



- Mark Container of Deionized / Distilled Water as Universal Cleaner.
- Loosen the Cap on the smaller chamber of Universal Cleaner Bottle.
- (40-810 960 ml Calibrated Bottle Illustrated.)

- Squeeze the center of the bottle gently and fill the measuring chamber to the desired level.

Recommended Dilution Ratio:

Deionized / Distilled Water.
3.8 Liter / 1 US Gallon
1.9 Liter / ½ US Gallon
0.95 Liter / 1 US Quart

Recommended Dilution Ratio:

Universal Cleaner Milliliter (ml)
30 ml Cleaner
15 ml Cleaner
7.5 ml Cleaner

- Pour measured Universal Cleaner into the Container of Deionized / Distilled Water.
- Replace Cap on Deionized / Distilled Water Container.
- Several quick shakes will completely dilute the Universal Cleaner.
- The Universal Cleaner has a stable shelf life for 5 plus years.